

Our #1 goal is to make sure

you are happy with our product.

If you have any comments or suggestions,

we would love to hear from you.

Please email us at:

sales@advancemobility.org

or call us at **800.241.1848**



8575 Magellan Parkway, Suite 1000 • Richmond, VA 23227
T: 800.241.1848 • F: 804.262.6277
www.advancemobility.org

**Advance
Mobility**[™]
EMPOWERING INDIVIDUALS

Freedom Footplate Instructions



ADVANCE MOBILITY

*Advance Mobility is a Division of
the Baby Jogger Company*

Freedom Footplate Installation

This footplate works only on the Freedom Mobility Device

Step 1

Rotate lower end tubes of Push Chair frame.




Step 2

Insert footplate onto the Push Chair frame.



Step 3

Secure footplate onto Push Chair frame using the enclosed screws. A phillip screwdriver will be needed.

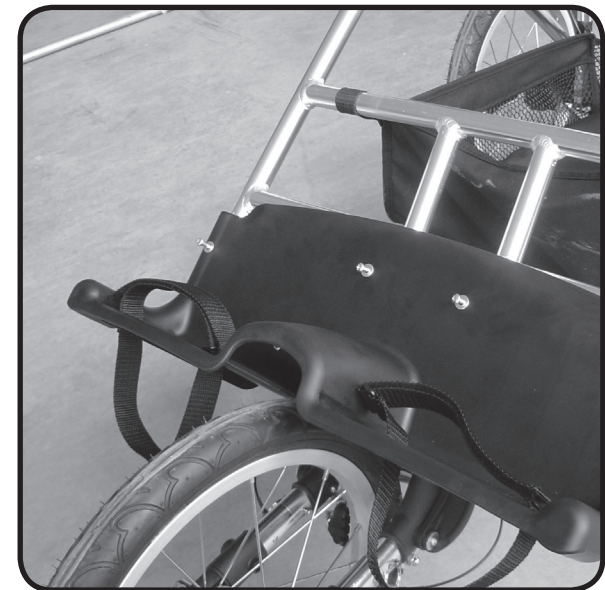

Note: Use tapered screws to secure the center of the footplate