

Assembly Instructions



ADVANCE MOBILITY

Freedom

*Important, Keep for
Future Reference*

For the sake of your precious cargo, please read these warnings before using your Push Chair.

- Important, keep this booklet for future reference.
- Accessories which are not approved by the manufacturer or distributor should not be used.
- ⚠️ • **WARNING:** Prevent serious injury to your passenger by always using the safety harness.
- ⚠️ • **WARNING:** Any load attached to the handle affects the stability of the stroller.
- ⚠️ • **WARNING:** Do not leave passenger unattended.
- Always use your wrist strap – these Push Chairs can really roll, and the wrist strap gives you control if you should slip or fall down.
- The passenger should remain seated at all times (no standing in the Push Chair).
- Watch out for cars – never assume that a driver can see you and your passenger.
- Never use Roller Blades while using the Advance Mobility Push Chair.
- Total weight load recommended for this Push Chair is 200 lbs. (combined passenger and baggage weight).
- This Push Chair seats one passenger.
- ⚠️ • **WARNING:** Recommended weight for seat pocket is 2 pounds and basket is 10 pounds. Excessive weight may cause a hazardous unstable condition to exist.
- It may be unsafe to use replacement parts other than those supplied or approved by the manufacturer.

Frame




- To assemble, simply unfold frame...


Frame



- Swing bottom frame tubes toward footplate.

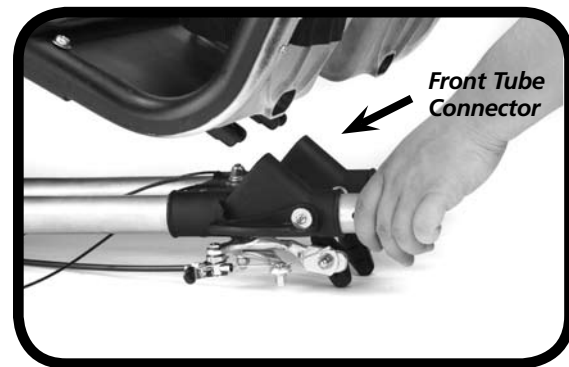


- Insert bottom of footplate into front tube connectors.

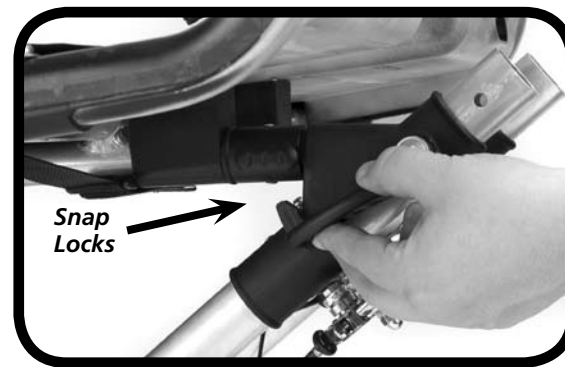

- Push up to secure snap locks on both tubes.

Frame

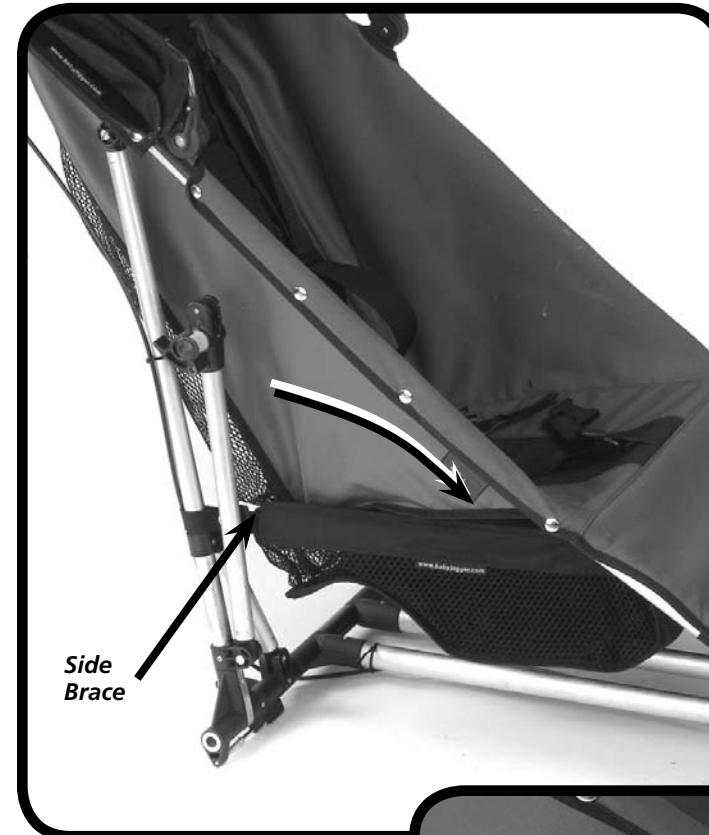

- Swing side braces down to mounting brackets.



- While pushing latch button slide side brace over mounting bracket, release when pin is aligned with hole in bracket.




- Secure side brace clamp.

Repeat on both sides.



Front Wheel

- 1** To install front wheel, lay Push Chair back so handle is resting on floor. Open the calipers by flipping the caliper lever toward the rear.
- 2** Insert the front wheel assembly into the front frame tubes until pin holes are aligned.
- 3** Insert wheel pin on each side.
- 4** Close caliper by flipping the caliper lever toward the front.

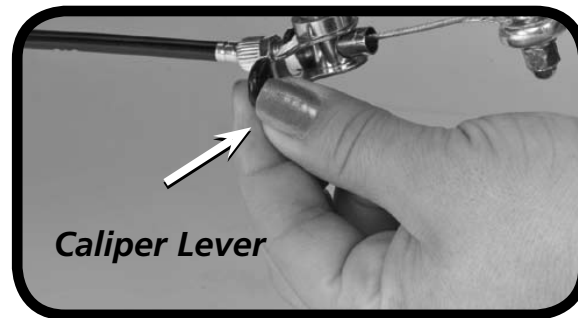

Front Swivel Wheel

- 1** To install front swivel wheels, lay Push Chair back so handle is resting on floor.
- 2** Insert the front wheel assembly into the front frame tubes until pin holes are aligned.
- 3** Insert wheel pin on each side.



Rear Wheels



- To install, gently slide the wheel's axle into the hole in the axle end until you hear it click into place.

- To remove, simply push in the spring lock while pulling out the wheel.



Reclining Seat




- Simply adjust seat straps to desired degree of recline.

- Pull on strap when you want to return the seat to its upright position.



Be sure to remove the board from the back of the seat before washing.

Safety Harness

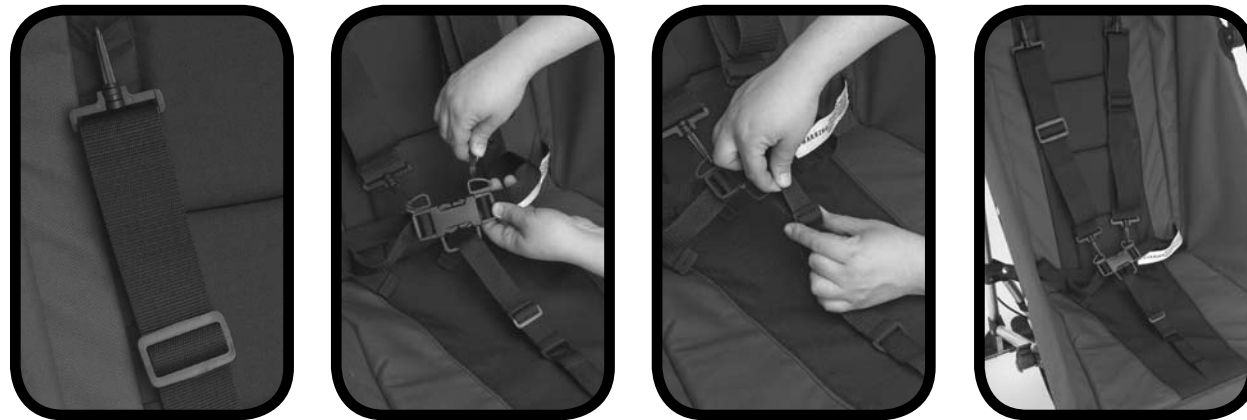

Safety harness:

- Clip shoulder straps to upper fabric loop that best suits passenger then attach lower clips to buckle bracket.
- Adjust center safety strap as necessary for passenger comfort.




WARNING: Avoid serious injury from falling or sliding out. Always use safety harness!

Hand Brake

Hand Brake with Parking Button

- To use the parking button feature, simply squeeze the brake handle and push the park brake button.

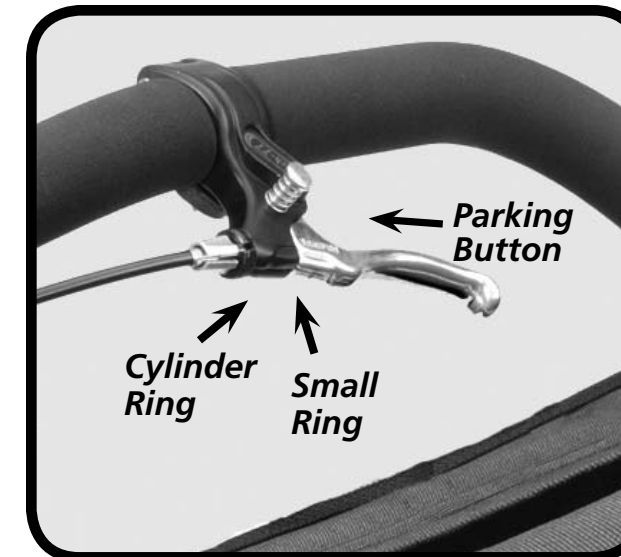


Adjusting the Brake

- If you don't have good braking action, simply squeeze the brake handle five or six times. If the problem persists, loosen the brake's cylinder ring to adjust the small ring, and then move the small ring away from the handle to tighten the cable. (Please do not over tighten!) To lock the rings, tighten the cylinder ring against the handle while holding the small ring in place.

Rear Parking Brake

Rear Wheel Parking Brakes

- Simply push brake pedal down to engage brake and lift up to release.

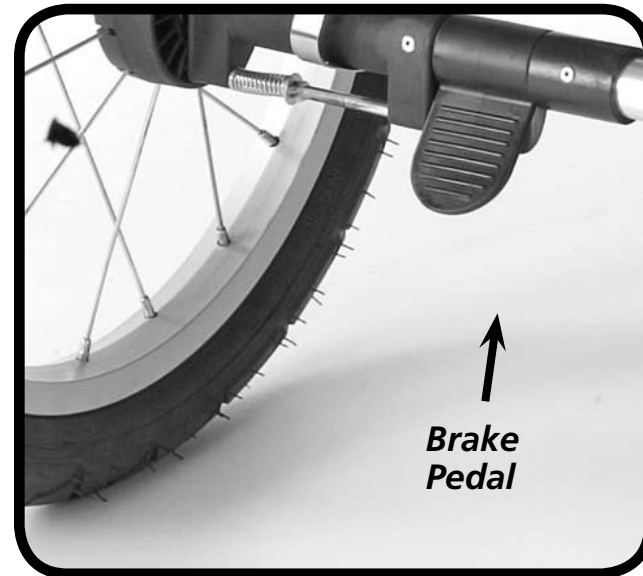

Canopy

Image shown with seat fabric removed for clearer viewing.

NOTE: Some Push Chairs are shipped with the Canopy, Seat and Basket pre-assembled. Assembly instructions are given if any pre-assemble items are removed for cleaning or replacement.



1 Using a phillips head screwdriver, screw brackets to Push Chair frame.



2 Snap to Push Chair frame.



3 Fasten upper velcro straps

In Motion

Steering

- Steering your Push Chair is simple: it's balanced to turn with just a light touch on the handlebar. For tighter turns, push down on the handlebar and raise the front wheel to steer.

Wrist Strap

- Make sure wrist strap is around your wrist whenever your passenger is in the Push Chair. The Advance Mobility Push Chair is lightweight and ready to roll – with or without you – and the wrist strap helps you gain control over your Push Chair.

In the Dark

- Since we are concerned about the safety of you and your passenger, we recommend that you not use your Push Chair at night. However, if you choose to, please attach reflective strips to the wheel spokes and the seat. Bicycle lights can also be mounted on the handlebar.



Push Chair Care

Washing

- You can machine wash your Push Chair's seat in cold water with a mild detergent. To minimize shrinkage, simply reattach the seat to the frame to dry. (Please do not put the seat in the dryer!)
- Be sure to remove the board from the back of the seat before washing.
- You may clean the Push Chair's frame using a soft, dry cloth, and an occasional spray of household wax will maintain its healthy glow.



Wheel Care

- Moisture – especially from the beach – can rust the wheel bearing case. Spraying regularly with a silicone lubricant will keep your wheels looking healthy.
- Also, if you use your Push Chair for large amounts of miles, your Push Chair will benefit from a yearly visit to the bike shop for wheel alignment. Refer to the number on the tire wall for the optimum pressure for your tires.

Storing

- Sunshine can fade seat fabric and dry out tires, so storing your Push Chair indoors will prolong its good looks. However, if you do store it outside, using a tarp will protect against weather wear.

Thank You!

We wish you many happy journeys with your new Advance Mobility! Your Push Chair is a great product, and we are proud to manufacture it. If you have any questions or comments, please give us a call at:

1.800.241.1848



Advance Mobility is a Division of Baby Jogger

8575 Magellan Parkway, Suite 1000 • Richmond, VA 23227

T: 800.241.1848 • F: 804.262.6277

www.advancemobility.org